


# Capri - Massage Chair

## OPERATION INSTRUCTIONS



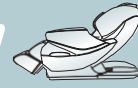



## Massage chair



Thanks for purchasing this product. Please read this manual instruction carefully in order to know how to use this product correctly. Please pay close attention to safety warning section and keep this manual for future use

Note: Our Company reserves the right to revise the design and description of this product without any further notice. Please refer to the actual color of this product

## CONTENTS

Safety and maintenance:	02
Name and function of components	04
Functions	05
Method of usage	06
1. Standard equipped	06
1.1. LCD manual controller control the massage chair	07
1.2. Switch the main power supply on	08
1.3. To Start a Massage	08
1.4. Shut off power and finish massage	12
2. iPhone or Android tablet system operate massage chair	13
2.1. Preparation before operation	13
2.2. Switch on the machine	14
2.3. Massage function control operation	15
2.4. Backrest frame and calf rest frame position adjust	22
2.5. Setting	23
2.6. APP download	23
2.7. Wireless blue tooth music player operation	24
2.8. End massage, power off	25
Arm assembly installation	25
The disassemble and using of head and back cushion	27
Floor protection	27
Moving the chair	28
Product specification	28



## Important Safety Warnings

- To prevent injuries, children and/or pets should not play around the chair when it is in operation.
- Users must stop and turn the power off immediately if they feel ill.
- If you feel that the chair is functioning abnormally, turn off the power switch and unplug the power cord. Call our tech support department immediately. **DO NOT** attempt to disassemble or repair the chair.
- Keep all body parts away from the internal mechanisms.
- **STOP** using the chair and **UNPLUG** the power cord if any part of the fabric cover is torn or worn-out causing exposure of the mechanisms. Please contact tech support immediately.
- Users with recently chemically treated hair or those using any topical hair products should place a towel between the hair and the upholstery in order to protect the material.
- It is not recommended that those under the age of 16 use the chair.
- People weigh over 300 pounds using the chair will void the warranty.
- iDo not use the chair if the back pad is flipped over exposing the rollers.
- iPlease use well-grounding power supply suited to this product.
- iPlease unplug after using or before cleaning to avoid injury or damage.
- iPlease operate this product according to this manual instruction.
- iDo not use accessories which are not recommended.
- iDo not use this product outdoors.
- iPlease read this manual instruction carefully before operation.
- i20 minutes of usage each time is recommended.
- iPlease do not fall asleep while using this product.



## Environmental Usage

- iPlease do not use the chair under high temperatures or in moist environments
- iPlease do not use the chair in transition between hot and cold environments. Let it return to room temperature for at least an hour first.
- iPlease do not use the chair in a dusty environment.
- iPlease do not use the chair where there is not enough space for use or where there is poor



## People who are not suitable to use this product

- iPersons with any other medical conditions in addition to those specified below need to consult their physician prior to use:
  - iOsteoporosis
  - iHeart Disease
  - iElectronic medical equipment such as a heart pacemaker
  - iFever
  - iPregnancy or women in menses
  - iSkin injuries or skin disease
  - iMental disease
  - iHeat sensitivity
  - iIrregular curvature of the spine or other spine abnormalities
  - iMalignant tumors
- iAdditional Safety Information:
  - iDo not use the chair if you are wet



## Safety

- ¡Check the voltage if it is suitable to the specification of this product.
- ¡Do not use wet hand to pull the plug.
- ¡Do not make water go inside this product to avoid electric shock or cause damage to this product.
- ¡Do not damage the wires or change the circuit of this product.
- ¡Do not use wet cloth to clean the electric parts such as switch and plug.
- ¡Be away from this product under power cut state to avoid injury if the power comeback suddenly.
- ¡This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- ¡Children should be supervised to ensure that they do not play with the appliance.
- ¡If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard.



## Maintenance

- LDamage caused by the user attempting to disassemble the chair or replacing spare parts themselves will void the warranty.
- ¡This product should be maintained by certain appointed agents, DO NOT attempt to disassemble or repair the chair yourselves.
- ¡Please do not forget to cut the power after usage.
- ¡Do not use this product if the socket looses.
- ¡If this product will be left unused for long, please curl up the wires and reserve this product in dry and dustless environment.
- ¡Do not reserve this product under high temperature or near fire and avoid longtime direct sunshine.
- ¡If the detachable cord / supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a Hazard.
- ¡Please clean this product by dry cloth. Do not use thinner, benzene or alcohol.
- ¡The mechanical components of this product are specially designed and made, it is not necessary to be specially maintained.
- ¡Do not use edged res to thrust this product.
- ¡Do not roll or pull this product above uneven ground, it should be lifted before moving.




## Solution of ordinary malfunctions

- ¡Solution of ordinary malfunctions: Solution of ordinary malfunctions
- ¡It is nature movement voice if the motor sounds while in usage.
- ¡If the controller cannot work normally, please check the plug and socket if they are connected strongly and make sure the switch is turned on.
- ¡If the rated working time is over, the product will automatically activate the power-off button; if this product works continuously for long, the temperature protector will make the product



## Name and function of components

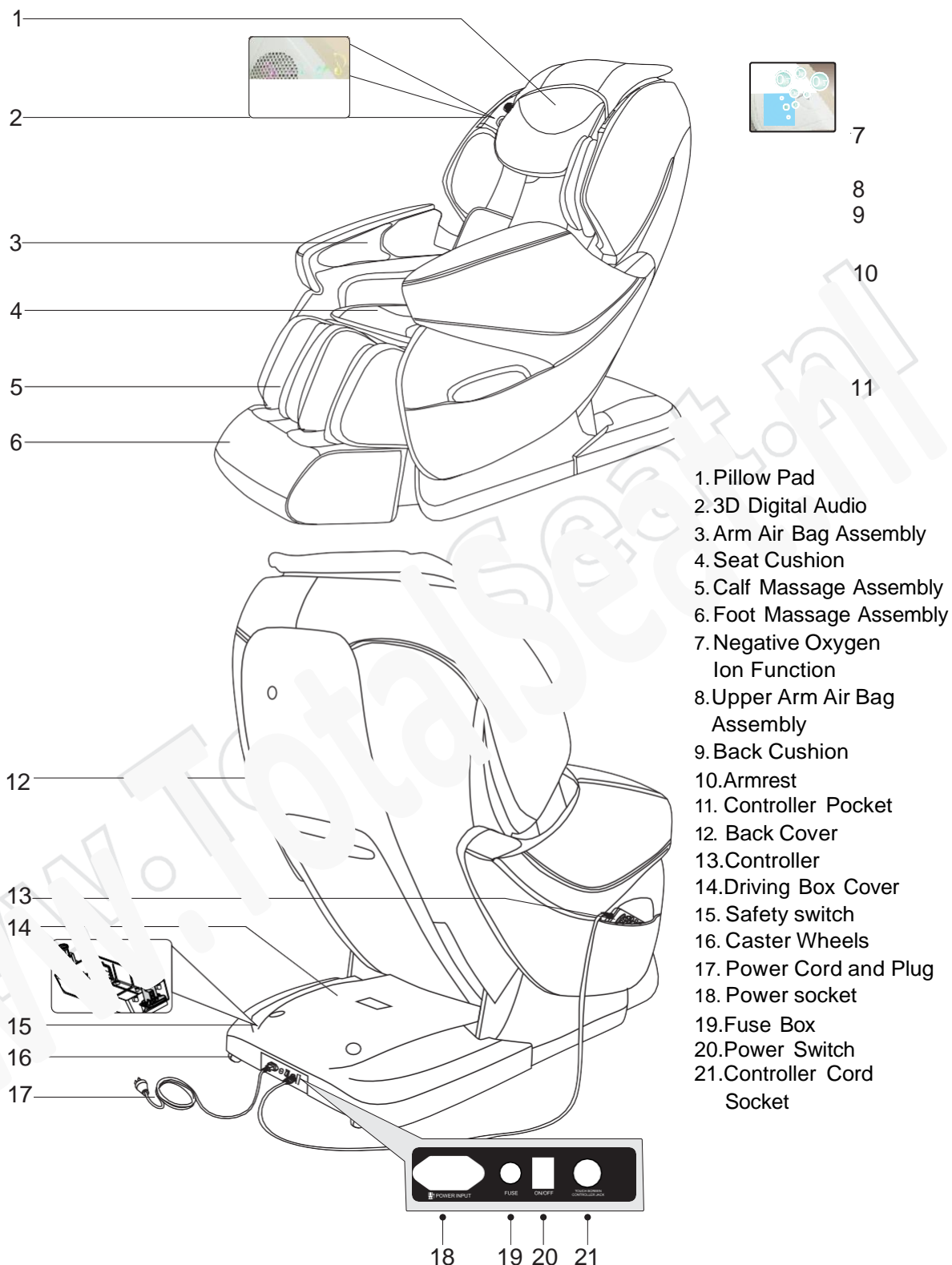



## Functions

- This product is designed with a set of smart 3D mechanical hands, which can move up and down, stretching back and forth, four-wheel driven with muted design.
- The shoulder part is designed with automatic detection and micro adjustment function; body curve and massage points automatic detection, it can automatically adjust the distance of massage hands' moving back and forth according to the detected body curve and massage points, to make the massage more humanistic and scientific.
- Comfort, ease, ache relieve, fast experience, relax, music sync, waist stretch, full air pressure and other specific massage functions.
- Set with two kinds memo massage functions M1 and M2.
- Manually select the upper body massage; three massage position options of overall, partial and fixed position; six massage methods of shoulder grasping, kneading, tapping, shiatsu, kneading & tapping and 3D; five massage speed levels available; under tapping and shiatsu massage status, the width of the massage hands can be adjusted in five levels.
- Arm air massage function (built-in 8 air bags), upper arm air massage function (built-in 4 air bags), with 3 intensity adjustments.
- Lower body air pressure massage function: back air pressure massage function (built-in 4 airbags), buttock air pressure massage (built-in 16 airbags), calf rest air pressure massage function (built-in 36 airbags), with 3 intensity adjustments.
- On upper arm has two grades width adjustment function When you adjust to the last grade, to push into the arm about a gear distance then lose your hand and arm device will automatically reset, and so on in order to adapt to the different Shoulder breadth of users.
- Back heating function, use carbon fiber as infrared heating source.
- The footrest can be extended to suit various heights.
- With slide rail structure on backrest, make a zero-space design to maximally save place.
- Calf rest lifting, backrest lifting and auto lying models adjustment.
- With negative oxygen ionizer on upper arm.
- Intelligent terminal (cell phone or tablet PC) control the massage chair by blue tooth.
- MP3 player function, upper arm built in 3D digital audio.
- Connect the smart device via bluetooth to play music .
- Auto massage program download.
- Place 8 natural energy magnet in calf airbags.





## Standard functions

### 1. Negative oxygen ion function (button on left upper arm).

- ¡Innovative negative oxygen ionizer design on head part, it can emit negative oxygen ion ceaseless.
- ¡By the national infrared and industrial electric products quality supervision and inspection center verification, its negative oxygen ion concentrations up to 240 thousand /cm<sup>3</sup>, equal to forest or waterfall, helpful for spontaneous recovery on human body.
- ¡Negative oxygen ion can reduce PM 2.5, optimize room air.

### 2. Magnet therapy function


- ¡Place 8 natural energy magnet in calf airbags.
- ¡By the national infrared and industrial electric products quality supervision and inspection center verification, the surface of the magnetic intensity is 73mT, according with the best protection strength of national standard £40-110mT£.

### 3. 3D Digital Audio


- ¡With 3D digital audio equipment on each side of the upper armrest, connect with cell phone music to play through, you can enjoy perfect fusion of music and massage when relaxing.




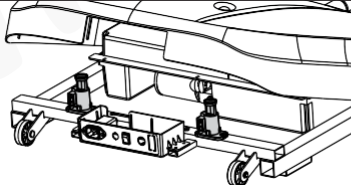
Negative oxygen  
ion function

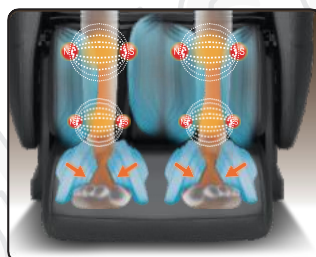


Magnet  
therapy function

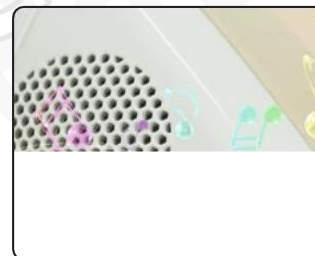

3D Digital Audio

 <b>WARNING</b>		With safety switch in the power source box, if any heavy weights on the box cover, the backrest lay down function will be stopped, avoiding accident crush injuries when the backrest down.
---	---	---



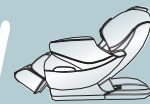

**LCD** LCD manual controller control the massage chair



NOTE: This is for reference only.



## Switch the main power supply on



Diagram of plug in

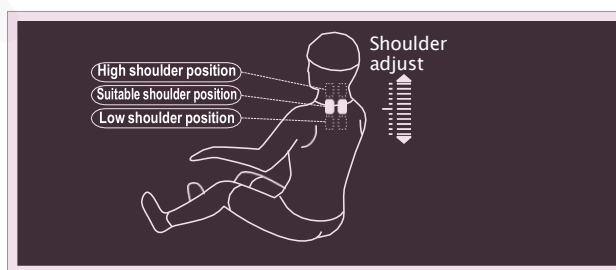


Diagram of the switch position of power supply

## To Start a Massage

- 1 Press the red power button to start the massage, the massage chair will automatically recline down.
- 2 Press certain auto massage button, the massage hands proceed auto detection of your shoulder and body scan, the calf rest roller begin to work at the same time (choose the auto program of rapid massage to start the massage program immediately, shoulder detection only to save half the time), if you made no choice of the auto massage program after 20 minutes, the massage chair will automatically switch off.
- 3 Upon completion of scan, a beeping sound will begin emitting from the controller and the massage heads will gently roll on your shoulder region to demonstrate where the chair has sensed the position of your shoulders. During this period, if you would like to further adjust the detected shoulder region, you can do so by depressing the up or down shoulder position buttons seen below, it has 11 grades.
- 4 If you have not made any adjustments to the shoulder position or selected a manual massage mode during the time after scanning, the chair will automatically start the Fine massage program for you.

Buttons	Description	On Display
 Shoulder position adjustment (Upper button)	Press to adjust the massage heads higher.	
 Shoulder position adjustment (lower button)	Press to adjust the massage heads lower.	





Choose massage methods

Memory message: When it is on shoulder position detection, you can use M1 or M2 memory button to start massage, no need to wait till detection finished, it will back to the stored massage models to massage.



Characterization of automatic massage

Buttons	Description	On Display
	Demo for all the massage function	<b>F0</b> Auto
	most comfortable massage experience	<b>F1</b> Auto
	depth massage, awaken body energy	<b>F2</b> Auto
	Depth massage on acupuncture point, relax ache area	<b>F3</b> Auto
	Waist, buttocks, arms, legs air bag work at same time, relax the whole body	<b>F5</b> Auto
	When the calf air bag work and squeeze calf part, the backrest lay down and calf rest pull down to stretch the waist part, then return to original position and repeat above actions, relieve waist fatigue and bounce back	<b>F6</b> Auto
	Music synchronization massage	<b>F7</b> Auto
	Relax your body	<b>F8</b> Auto


















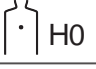
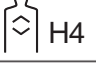
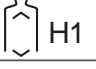
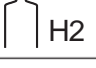
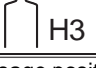


APP program auto calling

Buttons	Description	On Display
	APP Automatic function selection: 3 auto modes	A1~A3
	Confirm selection: If the current APP mode is invalid, press confirm button, the horn remind invalid.	A1~A3 <sup>App</sup> Auto














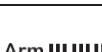
Upper body manual operation, lower body manual operation

Upper body manual operation









Buttons	Description		On Display
 SHOULDER GRASP	Shoulder grasp: auto turn to shoulder grasp massage, speed and intensity adjustable		
 KNEAD	Kneading: speed and intensity adjustable		
 TAP	Tapping: subdivide into two massage modes, speed, width, intensity adjustable		
 KNEAD&TAP	Kneading & tapping: speed and intensity adjustable		
 SHIATSU	Shiatsu: subdivide into two massage modes, speed, width, intensity adjustable		
 3D	3D subdivide into four massage modes, speed, intensity adjustable		
 SPEED	Speed: 5 grades adjustable		
 WIDTH	Width: 5 grades adjustable		
 BACK STRETCH	Fixed position massage		 H0
	Back and forth massage on partial position		 H4
	Back and forth massage on upper body		 H1
	Back and forth massage on upper back		 H2
	Back and forth massage on lower back		 H3
 LOWER ADJUST	Under fixed or partial position massage mode, massage hands can move downward	Massage hand move downward	Massage position adjustment (under fixed and partial position massage mode, it is adjustable)
 UPPER ADJUST	Above fixed or partial position massage mode, massage hands can move downward	Massage hand move upward	




### ;Air compression massage functions

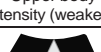Buttons	Description	On Display
 LOWER BODY	Lower body air compression function (total 3 modes: lower body, back and waist, calf )	Feet  Back  Buttocks 
 INTENSITY	Lower body, back and waist, calves air compression intensity adjustable, 3 grades adjustable	Feet  Back  Buttocks 
 ARTHROSIS TRACTION	Under lower body air compression mode or calf air compression modes, switch on/off calf rest stretch function	
 ARM AIR PRESSURE	Arm air compression massage on/off and intensity adjustment, 3 grades adjustable	Arm 



### ;Other functions

Buttons	Description	On Display
 Upper body intensity (weaken)	Back massage intensity weakened: massage hands move to any position, can weaken the current intensity	
 Upper body intensity (strengthen)	Back massage intensity strengthened: massage hands move to any position, can strength the current intensity	
 SOLE ROLLER	Foot roller switch on/off and speed adjustment, 3 grades adjustable	
 HEATER	Back far infrared heating therapy, feel warm after 3 minutes	

### ;Massage Time Adjustment

Buttons	Description	On Display
 TIME	The default time for massage is 20 minutes, max work time is 40 minutes. Press the time button to add in 5 minute increments. You can set the time range anywhere from 5 minutes to 40 minutes.	20Min

### ;Negative oxygen ion generator Controller button is in the upper arm

Buttons	Description	On Display
Negative oxygen ion function	Negative oxygen ion function on & off	
Speed	2 grades available	



The angle of backrest and calf rest


Buttons	Description
	Auto lay down mode : with 3 auto lay down mode , when press the button, turn into one mode, follow by cycle.
	Calf rest lifting button: press this button, calf rest lifting slowly, stop when release it.
	Calf rest decline button: press this button, calf rest decline slowly, stop when release it.
	Backrest lifting button: press this button, back rest lifting slowly, stop when release it.
	Backrest decline button: press this button, back rest decline slowly, stop when release it.



### Shut off power and finish massage

Press power button during massage to stop massage function immediately or one setting massage program finished, backrest and footrest won't return to original position, and the remote controller display screen will blink.

- Press M1 or M2 button within 10 seconds to return to massage mode.
- Or press the power button Within 10 seconds, the massage hands, backrest and footrest will return to original position automatically.
- Or waiting for 10 seconds, the massage hands will return to original position, but backrest and footrest won't return to original position.

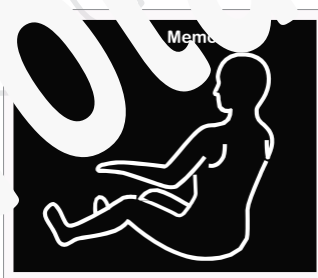

Turn off power

Figure (Cut off the power of the machine)


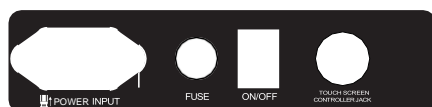


Diagram of the switch position of power supply




Method of turn off power

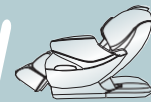

## iPhone or Android tablet system operate massage chair

### Application program interface

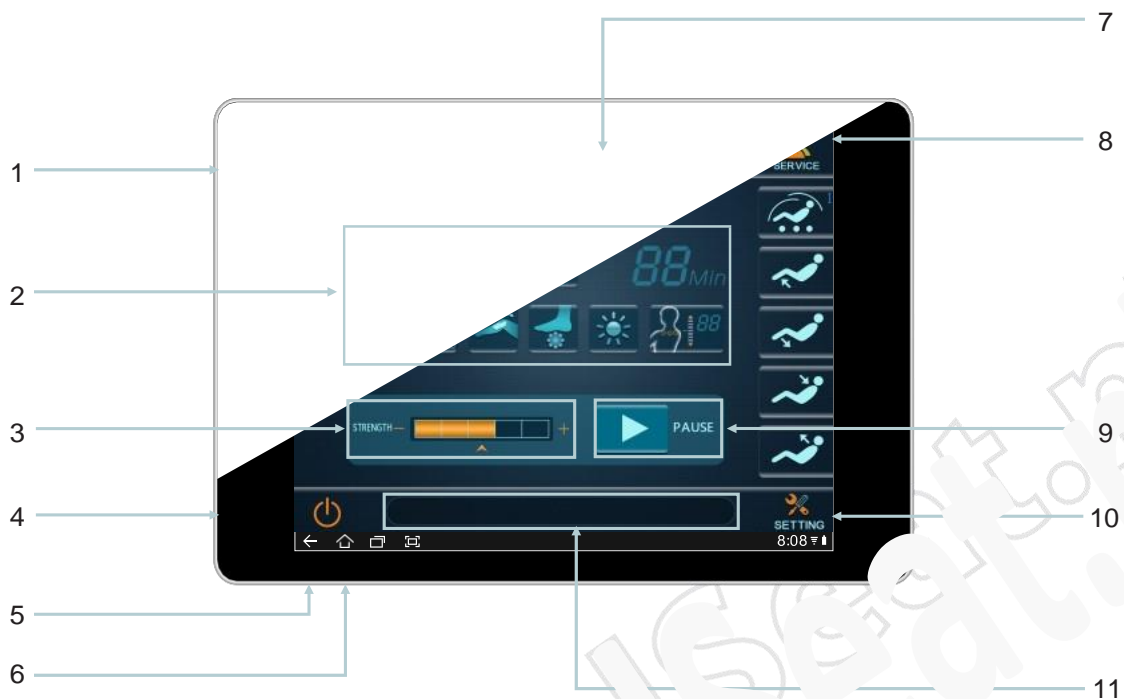

Number	Description	Number	Description
1	function keys	7	status-bar
2	massage status information display area	8	Service
3	upper body intensity adjustment	9	operation/pause key
4	massage button	10	function setting function
5	long press for 2 seconds to exit program	11	information prompt line
6	minimize program		

## Preparation before operation

### 1) Installation program download and install

Installation program supplied by agent or download from the network provide by agent .





- USB connect the tablet and computer, install the program in the tablet through the software tools (such Peasecod, 360 mobile phone assistant). After install successfully, the program icon will be shown in tablet application program. Any questions, please consult the tablet supplier when install the program.



Install successfully, a (application program) icon appears.

## 2) Blue tooth link

- Turn on the chair power switch .

- Open the applicant menu in the tablet main screen, choose setting to open the setting interface.
- Operate blue tooth function ?choose blue tooth setting? click and check attachment equipment, when find *Fihonest* icon, click it to match tablet blue tooth with massage chair blue tooth(matching code: 0000). When match successfully, click *Fihonest* again to connect tablet computer blue tooth with massage chair blue tooth, when connect successfully, application program can be used to operate massage chair.

## 3) The blue tooth module abnormal problems and solutions

- After the blue tooth paired succeed, if it cannot connect well with the blue tooth of massage chair, please try to cancel the blue tooth pairing, turn off the power of massage chair, and restart it, then re-connect as above.
- The blue tooth show connect succeed, but in to the application program, cannot control the massage chair well, or the information tips showed blue tooth connect failed, it shows abnormal in blue tooth module, you should exit the APP and connect the blue tooth module again, or turn off the power of massage chair, and restart it, then re-connect as above.

## Switch on the machine

- Make sure the massage chair power switch is off, connect the power cord with the massage chair, plug it into the good ground connection socket, turn the power switch on, the massage chair into start up states.



Diagram of plug in


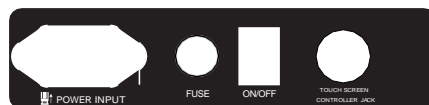



Diagram of the switch position of power supply

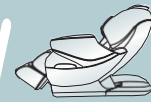

## Massage function control operation

### 1. To start a massage

¡In pad screen, unfold the APP listing, click  to start up the massage chair control program.

¡First into startup interface after you start up the program, after 5 seconds into massage information interface.

¡Click ¡power on/off¡ on the display interface to start massage function, massage chair lay down automatically



Startup interface



In standby mode massage information display interface

### 2. Shoulder position adjustment

¡Automatically detecting shoulder position and acupressure points, please wait till detection finished.



¡It sounds ¡di-di-di.....¡ after finished the detection and into shoulder position adjust interface. Now, you can adjust the shoulder position by pressing up or down button, it has 11 grades available. If you have not made any adjustments to the shoulder position or selected a manual massage mode within 10 seconds after scanning, the chair will automatically start the ¡Fine¡ massage program for you.




Shoulder position and massage points detection interface



Shoulder position adjust interface

Shoulder position adjust button	Function description
	Press to adjust the massage heads higher
	Press to adjust the massage heads lower

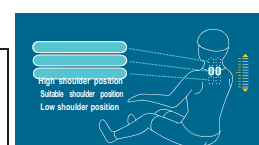



### 3. Massage function running and pausing

In massage information interface, when the massage chair in running state, press running /pausing button, all the massage function will stop, and into pausing state; in pausing state, press running /pausing button, the massage function will continue to running as paused before. All the massage function turn off automatically if the pausing time over 10 minutes.



running state interface



pausing state interface

Pause/start	Function description
PAUSE	Press this button the massage function continue to running as be paused before.
RUNNING	Press this button, pause all the massage function

### 4. Upper body massage intensity



In running state, you can adjust the massage hands' intensity on the back of upper body by click the intensity adjust icon in the massage information interface, 5 grades available.



INTENSITY					Strength: grades adjustable	
1	2	3	4	5		










## 5. Auto message

- Click  popup message function selecting menu, choose what you need.
- Click  into auto function selecting interface, click auto icon choose auto message mode. In auto message state, the speed and width are nonadjustable.




Auto message function selecting interface


Button	function description
	Most comfortable massage experience
	Relax your body
	Depth massage on acupressure point, relax ache area
	Neck and shoulder, waist, buttocks, arms, legs air bag work at same Time, relax the whole body
	Depth massage, awaken body energy
	When the calf rest air bag work and squeeze calf part, the backrest lay down and calf rest pulls down to stretch the waist part, then return to original position and repeat above actions. Meantime the seat air bag and massage hands work together.
	Back to running state information interface



## 6. Manual message


Click  into manual message function selecting interface, select manual message methods, back stretch mode, speed, width function. In flapping, 3D, shiatsu, stop state, the width of massage hands is adjustable. Except the stop state, the speed is adjustable in any other massage methods








### 1) Manual message - methods, speed, width

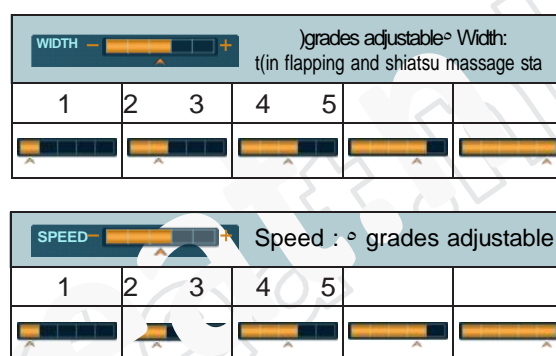In the interface of manual function selecting, click the  into message technique selection, speed, width adjustment interface, click the corresponded technique icon, speed icon, width icon, choose the personalized massage function.

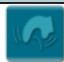


Auto massage function  
selecting-technique interface



Technique buttons	Description
 SHOULDER GRASP	Shoulder grasp: auto turn to shoulder grasp massage, speed and intensity adjustable
 KNEAD	Kneading: speed and intensity adjustable
 FLAP	Tapping: subdivide into two massage modes, speed, width, intensity adjustable
 KNEAD AND FLAP	Kneading & tapping: speed and intensity adjustable
 SHIATSU	Shiatsu: subdivide into two massage modes, speed, width, intensity adjustable
 3D MESSAGE	3D subdivide into four massage modes, speed, intensity adjustable
	Return to the running state information interface



## 2) Manual massage-stretch mode

In manual function selecting interface, click  into stretch mode interface, click the corresponding stretch mode icon choose your personalized massage function.








Manual massage  
function select-stretch mode




### Note

At the shoulder grasping mode, stretching mode selection is invalid.

Stretch Mode	Function description
 Whole back	Massage up and down on whole back
 Upper back	Massage up and down on upper back
 Lower back	Massage up and down on lower back
 Partial	Back partial massage up and down in a small area
 Fixed point	Massage on fixed point

## 7. Air compression massage

Click  into the air compression massage function selecting interface, click the corresponding air compression massage function icon. It include lower body air compression massage (lower body, waist, buttocks) and arms air compression massage.



## 1) Air compression massage-arms



STRENGTH <span>Intensity: grades adjustabl</span>		
1	2	3
In arm air compression mode, adjust the intensity, 3 grades available.		

Air compression massage-arms interface

Air compression function	Description
Arms	Switch on or off the arms air compression massage function

## 2) Air compression massage- lower body, waist, buttocks, legs stretch




STRENGTH <span>Intensity: grades adjustabl</span>		
1	2	3
In a state of startup of lower body, waist and buttocks, foot part air compression, you can adjust the air compression intensity, 3 grades available.		

,Lower body, waist  
buttocks, legs, legs interface



Air compression function	Description
LOWER BODY	Switch on or off lower body air compression massage function. Lower body air compression including waist, buttocks and foot part.
WAIST, BUTTOCKS	Switch on or off waist, buttocks air compression massage function.
FOOT PART	Switch on or off foot part air compression massage function.
LEGS STRETCH	Switch on or off legs stretch function. When you switch on lower body air compression or foot part air compression massage function, the default startup function is legs stretch.

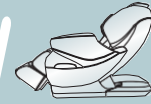




## 8. Other massage function

### 1) Other massage function-thermal therapy function


Press  button to enter massage function interface, press  button to start or stop backrest infrared heating function

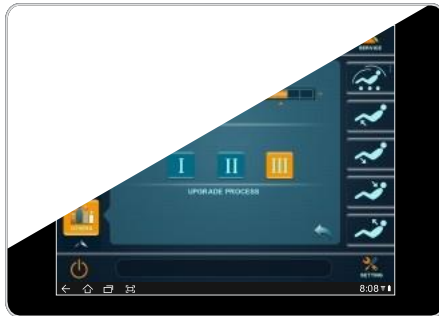



thermal therapy function interface

#### :Notes

minutes You will have warm sensation after when you switch on






Buttons	Function description
 <b>Thermal therapy Function</b>	Switch on or off thermal therapy massage function.


### 2) Other massage function- sole roller

Click  to the other massage function interface, click  to switch on or off this function. When the sole roller massage function in startup state, click sole roller speed icon, you can adjust the speed of the roller, with 3 grades optional.







Sole roller massage interface

SPEED   intensity: with grades adjustable		
1	2	3
		

Buttons	Function Description
 <b>SOLE ROLLER</b>	Switch on/off roller on sole massage function.



### 3) Other function massage-Updated program working



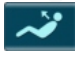

- Open application menu  , choose  service  into auto-massage program update interface .
- App auto-function choice , with  APP auto modes .
- Confirm your choice, the speaker will alarm if the selected APP is invalid.




Updated program working



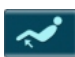

### Backrest frame and calf rest frame position adjust

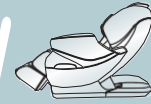Buttons		Function Description
calf rest go up and down		Click this button, calf rest rise up slowly, stop when release it
		Click this button, calf rest go down slowly, stop when release it
back rest go up and down		Click this button, back rest rise up slowly, stop when release it
		Click this button, back rest go down slowly, stop when release it
Lie down		nmodes for automatically lie down, every time with one Total with mode when press the butt





## Settings

### language SeleCtiOn

Click  icon into setting function interface, then click , popup language list box, select language type, the system with 3 language (simplified Chinese, traditional Chinese, English).

### volume adjuSt

Click  icon into setting function interface, then click , volume adjust icon, click volume + or volume - to adjust the volume of the music.




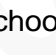
Setting function- language selecting interface




setting function- volume adjust interface


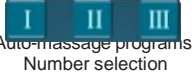




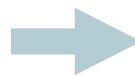

## App download











- Massage chair in standby mode, confirm the intelligent terminal (smart phone or tablet) in good connection with the blue tooth of massage chair.
- Open application menu , choose  into auto-massage program update interface.






Buttons	Description
 Start/exit	Start or exit app upgrade downloaded function
 Auto-massage programs Number selection	Auto massage program No. Choice
 OK	Confirm download address
 Testing	Intelligent terminal and the blue tooth of massage chair are in good connection
 Path	Choose the downloaded app document ( document. Bin )
 Upgrade program	Start to download app document

· **For example : Currently required download App auto program  irestAuto1.bin in A2 mode .**

- ¡ Confirm the intelligent terminal (smart phone or tablet) in good connection with the blue tooth of the chair in standby mode.
- ¡ Open application program  click service  enter into auto massage program updated interface .
- ¡ Click on/off  into auto program download number choice , click  choose auto-massage program number ( Remote controller screen display A2 and App ) , click confirm  again.
- ¡ Click connect test  test the connection between intelligent terminal and blue tooth of chair. Prompt window display test successful when the connect  is right , otherwise , it will display test error .
- ¡ After test successfully, choose firmware path  document , click scan  to choose irestAuto1.bin
- ¡ The last step is click firmware update  into auto massage program update, the prompt window display download successful ( remote controller screen display OH ) if updated finished , otherwise the it display download error to

### Wireless blue tooth music player operation

- ¡ Audio devices with blue tooth function (such as mobile , MID, tablet ) match with the chair blue tooth. The music transferred by the wireless blue tooth and played in the chair speaker.
- ¡ Audio device match with massage chair through blue tooth , please refer to above.

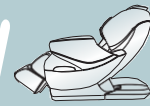

## End massage, power off

When the massage function is in a state of start-up, click **message on/off** will turn off all the massage function.

- Turn off power

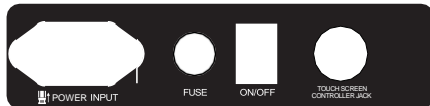


Diagram of the switch position of power supply



Method of turn off power



The power cord can plug from the power outlet only after the chair power switch turn off

## Arm assembly installation

- Find out the plug respond to the arm rest on the bottom of the seat frame, and through out from the square hole which on the fixed steel board.(figure 1)
- Draw out the plug from the square hole on the steel board, respond to every connecting plug( figure 2)

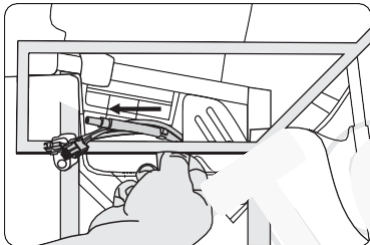


Figure 1

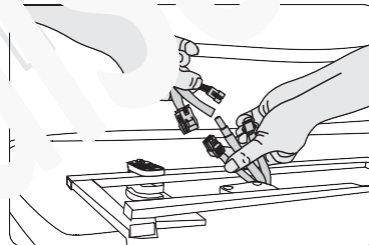


Figure 2

- Draw out the plug on the bottom of the seat frame and connect with the arm rest joint, air pipe responded to the right place (figure 3)
- After connect the responded connector, insert it gently through the square hole on the bottom of the seat frame, be careful not to fold the air pipe (figure 4)

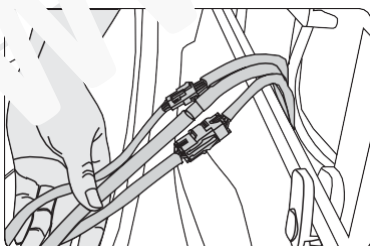


Figure 3



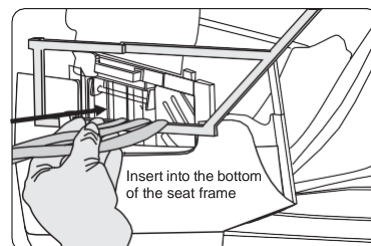


Figure 4

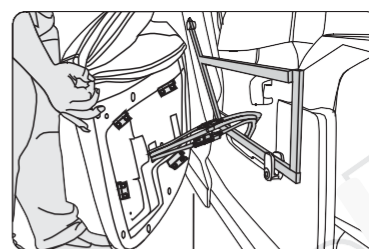

5. As figure, the hook on the arm rest respond to the box on the fix board, the steel board respond to the groove of arm rest.(figure 5)
6. Hold the arm rest with your hands, the back armrest is higher than the front, front armrest pin respond to box on the front part of the steel board , ( check the connect wire if it is inserted in to the bottom of armrest frame, avoid to get stuck), back armrest pin respond to box on bottom part of the steel board in turn, press down the back part of armrest to fixed two armrest top pin on the steel board. (figure 6)

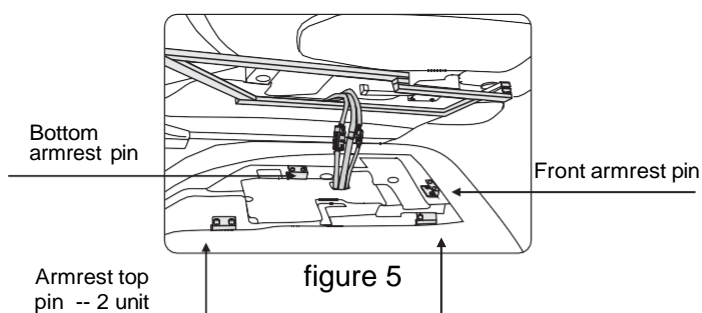

Note: the connect wire must be insert into the right place, avoid to get stuck.

7. Mount the arm rest to chair frame, the mounts for the arm should slide into their respective locations on the frame of the chair without applying force, press down slightly on the arm to lock into place. then press the armrest make sure the pin into the box on the fixed steel board, and pull it with your hand to check it is fasten tightly.(figure 7)
8. Find out the install hole on the front part of the fixed steel board, screw the screws with hex wrench.( Figure 8) .
9. Find out the install hole on the behind part of the fixed steel board, screw the screws with hex wrench, finish the installation, same installation of the two armrest. (figure 9) .

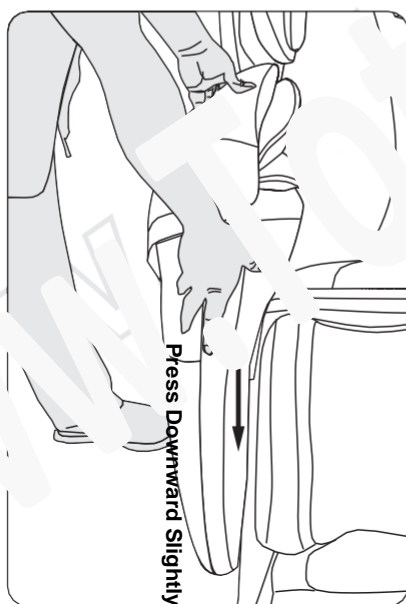


figure 7



figure 8

Figure: hex head screw fixed in front part of armrest

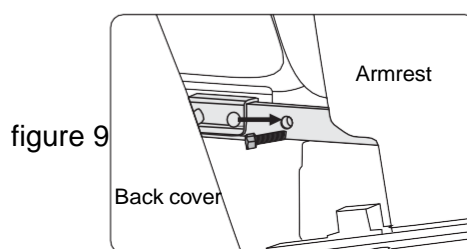


figure 9

figure: hex head screw fixed in back part of armrest



### The disassemble and using of pillow pad and back cushion

- Using head cushion can reduce knead massage strength for neck and shoulder, it all depends on you whether to use it (we suggest you using it). Head cushion and backrest are connection through a zipper (figure 1), upturn the back side of the head cushion, and put the AB flannelette inside of the head cushion insert the belt above the backrest (figure 2). Turn down the head cushion, put the back side lay on the backrest. It can adjust to a suitable position based on different size (figure 3).

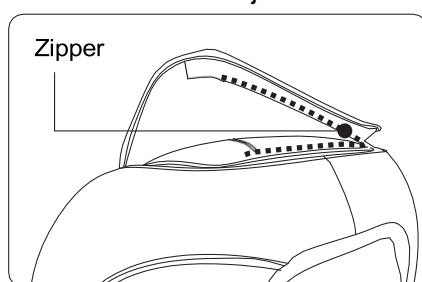


Figure 1

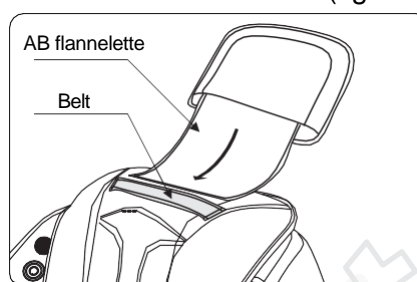
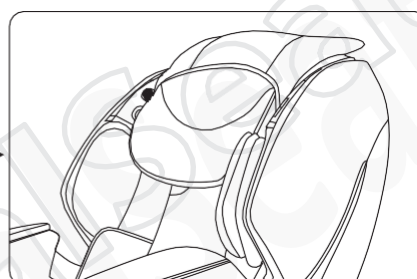


Figure 2

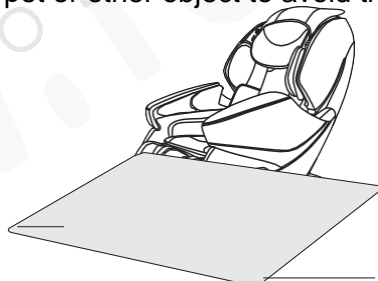


Figure 3



### Floor Protection

- Put the heavy massage chairs on the wood floor for a longtime may damage the floor, so please place a carpet or other object to avoid this damage.



**NOTED**

Do not move the chair with its wheels on wood floor, rough ground or in a narrow space. Two people are required to lift the chair in this case, carry the calf front sealing plate and back cover ( please don't lift the armrest ) .





## Moving the chair

Make sure all the wires are away from the floor, if you need to move, focus on the groove of the back cover, lift up to an angle (when the centre of gravity fall on the front caster), you can move the chair forward, then restore the chair to the normal angle .



**NOTED**

Please must turn off the power switch, then unplug the power cord and remote control wire before moving the chair .



## Product specification

Model: Capri

Description: Massage Chair

Configure: 01

Rated voltage: 110-120V ~ 60Hz ☐

220-240V ~ 50Hz/60Hz

Rated power input: 240W

Rated time: 20 Min

Noise: j58dB

Standby power: £0.5W

Safety Design: Class ø

Material introduction: PVC, PA, steel parts and electric &ele-ctron parts

MATERIAL	PU	LEATHER	CLOTH	WOOD





**SAVE THESE INSTRUCTIONS**

www.TotalSeat.nl